Art.: 6007

Model: DT-8836

Infrarot-Stirnthermometer | kontaktlos

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Art.: 6007

Model: DT-8836

**Infrared Forehead Thermometer |
Contactless**

**Please carefully read the operating
instructions before use!**

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E-mail: mail@bgs-technic.de | Web: www.bgstechnic.com

1. Introduction

The product is a thermometer for measuring the forehead temperature of the human body. The test results vary depending on the skin condition.

2. Precautionary measures

- Read the operating instructions before switching on the device.
- The operating temperature is between 10 °C and 40 °C.
- Keep this product away from current.
- Do not store this product at temperatures above 50 °C or below 0 °C.
- Do not use the product in moisture areas higher than 85%.
- Keep the product away from electromagnetic areas (e.g. radios, mobile phones, etc.).
- The product must not be exposed to sunlight, be left near to a fire and must not be immersed in water.
- Never knock against the product. Do not use the product if it has been damaged.
- The measured value may be influenced by sweat on the forehead, a cap or scarf. Make sure that the measuring distance is between 5 cm and 8 cm.
- For a more accurate measurement value of the body temperature, before the

measurement, the forehead thermometer must be placed on both sides for 15 to 20 minutes if the ambient temperature changes too quickly.

- If sweat is on the forehead or an abnormal body temperature has been caused for other reasons, please measure the temperature behind the earlobe.
- If the product has to be cleaned, wipe the surface lightly with alcohol.

Note:

- Before taking the measurement, remove any sweat and hair from the forehead.
- Use of the thermometer is not intended as replacement for consultation of your doctor.
- Please contact your dealer if you have any problems. Do not attempt to repair the thermometer by yourself.

3. Functions

Two types of selecting the measuring mode: body temperature and surface temperature

Display options in degrees °C and °F.

Acoustic notification possible in event of high body temperature.

Can store 32 sets of measurement data.

With date storage and automatic Power-Off.

Automatic range selection, resolution of the LCD display: 0.1°C (0.1°F).

LCD display with backlight

4. Others

The product is a thermometer for measuring the forehead temperature of the human body. The range of the normal temperature at different measuring positions:

| Measuring points | Normal temperature °C | Normal temperature °F |
|------------------|-----------------------|-----------------------|
| rectal | 36.6-38.0 | 97.8-100.4 |
| oral | 35.5-37.5 | 95.9-99.5 |
| at the armpits | 34.7-37.3 | 94.4-99.1 |
| on an ear | 35.8-38.0 | 96.4-100.4 |

The body temperature changes at different times of the day. It can also be influenced by outer conditions, such as age, gender, skin colour, etc.

5. Normal temperature range for different age groups:

| Age | Normal temperature °C | Normal temperature °F |
|-------|-----------------------|-----------------------|
| 0-2 | 36.4-38.0 | 97.5-100.4 |
| 3-10 | 36.1-37.8 | 97.0-100.0 |
| 11-65 | 35.9-37.6 | 96.6-99.7 |
| >65 | 35.8-37.5 | 96.4-99.5 |

6. Components

1. Infrared probe
2. LCD screen
3. Mode
4. Down
5. Up
6. Switch for the backlight
7. Trigger
8. Battery cover

7. Screen description, displays

1. Adapted mode (body temperature mode)
2. Direct mode (surface temperature mode)
3. Results
4. Alarm signal
5. Temperature °C / °F
6. Warning, low battery
7. Storage
8. Checking saved data

8. Technical data:


| | |
|----------------------------|--|
| Resolution | 0.10 C(0.1°F) |
| Ambient temperature | 10-40°C(50-104°F) |
| Storage temperature | 0-55°C (32-121 °F) |
| Rel. humidity | 20%~85% <i>m</i> W |
| Storage humidity | =<90% |
| Power supply | DC9V(1x6F22) |
| Dimensions | 150x122x43mm |
| Weight | 228g |
| Measuring range | Body temperature mode: 32.0 - 43.0°C Surface temperature mode: 0.0 - 100.0°C |
| Accuracy (body) | 32.0-34.9°C/89.6-94.8°F ±0.3°C/0.6°F 35.0-42.0°C/95.0-107.6°F ±0.2°C/0.4°F 42.1-43.0°C/107.8-212.0°F ± 0.3°C/0.6°F |
| Accuracy (surfaces) | 0.0-100.0°C/32.0-212.0°F ± 1.0°C/2.0°F |
| Measuring distance | 5 to 8 cm |
| Automatic power-off after: | Approx. 15 seconds |
| Measuring position | Middle of the forehead |
| Power consumption | <80 <i>m</i> W |

9. Description of the symbols

| | |
|---|--|
|  | DC 9V 1x 6F22 |
|  | If you want to dispose of the thermometer, do not mix it with general domestic waste. There is a separate collection system for used electronic products in line with the legal guidelines of the WEEE Directive (Directive 2002/96 / EC), which is effective in the European Union. |
|  | The device complies with ROHS 2011/65 / EU |
|  | Material: PVC |
|  | Keep dry and protect against moisture and water |
|  | Manual |
|  | This icon indicated instructions or instructions that must be followed in order to avoid damage to the use or other persons. |

10. Operation

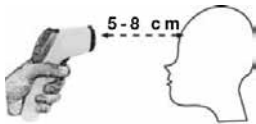- a) Insert the batteries
- b). For the first time, wait 10 minutes
- c). If it is not used for a long time, the thermometer detects the ambient temperature first so that switching on may still take one or two seconds.
- d). Focus against the forehead and hold at a distance of 5 to 8 cm. Then press the button to start the test. The test data are displayed within one second. Make sure that no hair, sweat, cosmetics or hat cover the forehead. (See figure below)

11. Setting

- a). Setting the temperature format --- F1

Keep "MODE" pressed for 3 seconds until F1 is displayed on the screen. Press ▼, to select °C and ▲ for °F

- b). Alarm temperature setting --- F2

Keep "MODE" pressed for 3 seconds until F1 is shown in the screen display and then press "MODE" again to access F2. Then press ▲ to increase the alarm temperature by 0.1 °C (0.1 °F) and press ▼, to reduce it by 0.1 °C (0.1 °F). Attention: The default alarm temperature is 38.0 °C (100.4 °F)

- c). Buzzer ON / OFF --- F3

Press the "MODE 3" button for 3 seconds until F1 is displayed on the screen. Then press "MODE" twice to call up F3. Thereafter, press ▼ to deactivate the buzzer (a sound icon disappears) and press ▲ to activate the buzzer (a sound icon is shown on the LCD screen).

- d). Switching off

Press and hold the "Light icon" button until the device is switched off.

This product is specialised for testing the body temperature of humans so that the temperature range displayed is between 32.0 °C and 43.0 °C (89.0 °F and 108.0 °F).

- e). Setting the measuring mode

Press the "MODE" button for the human body and conversion of the surface mode. Under the mode of the Surface temperature measurement, the range is: 0.0 ~ 100.0 °C (32.0 °F - 108.0 °F)

f). Save data

The test data are saved automatically. You can check the history results by pressing ▼ and ▲.

G). Change the battery

If the LCD display indicates a status that is too low, the battery must be changed.

Work step:

1. Open the battery compartment and take out the old battery.
2. If the thermometer is not being used for a longer period, take out the battery. When inserting the battery, observe the pole direction otherwise the device may be damaged. Rechargeable batteries are not permissible for this device.
3. Service life of the device
approx. 40,000 tests
4. Protection category against electric shock: type BF

12. Warning

- The glass on the screen is highly sensitive and fragile. Please be careful when using it.
- Do not use any rechargeable batteries.
- Do not hold it under too much sunlight and keep it away from water.

13. In this way, you will keep the device in a good state, and clean

- The most important part is the scanner head. Please protect this particularly.
- Clean the surface with 70% alcohol cotton wool.
- Otherwise, keep the thermometer away from all fluids.
- Keep the device in a dry location and avoid dust and contamination or sunlight.

14. Repairs

If you experience problems when using, follow the instructions below for solving:

The LCD cannot display the data.

If the body temperature is below 32.0 degrees Celsius or above 43.0 degrees Celsius, no data is shown on the screen. Only "LO" or "HI" is displayed

LCD display "HI"

This means that the temperature is above 43.0 degrees Celsius.

LCD display "LO"

This means that the temperature is below 32.0 degrees Celsius

There are several reasons as to why "LO" and "HI" are displayed as reference:

| Reason for "Lo" or "Hi" | Action |
|--|---|
| Influenced by hair or sweat | Remove the hair and sweat |
| Wind | Use only in a quiet environment |
| Distance too far | Reduce the distance (5 cm - 8 cm) |
| With strong changes in temperature (e.g. when moving from indoors to outdoors) | Please wait 20 minutes until the body has adapted to the new temperature. |

Declaration for electromagnetic compatibility:

electromagnetic compatibility and medical electronic - Please observe these when operating this device and follow the instructions.

ENVIRONMENTAL PROTECTION

Recycle unwanted materials instead of disposing of them as waste. All tools, accessories and packaging should be sorted, taken to a recycling centre and disposed of in a manner which is compatible with the environment.

DISPOSAL

Do not dispose battery in household waste.

Batteries should be disposed of in a responsible manner, they must be disposed at appropriate collection point.

Dispose of this product at the end of its working life in compliance with the EU Directive on Waste Electrical and Electronic Equipment. Contact your local solid waste authority for recycling information or give the product for disposal to BGS technic KG or to an electrical appliances retailer.

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